0123

ConvaTec

ConvaMax™ SUPERABSORBER

Superabsorber adhesive dressing / Superabsorber adhäsiv / Pansement superabsorbant adhésif / Medicazione Super-assorbente adesiva / Apósito súperabsorbente adhesivo / Superabsorbčné adhezívne krytie / Υπεραπορροφητικό συγκολλητικό επίθεμα / Superabsorberend adhesief verband / Pensó/ Curativo adesivo superabsorvente / Opatrunek z superabsorbentem, przylepny

INSTRUCTIONS FOR USE / GEBRAUCHSANLEITUNG / MODE D'EMPLOI / ISTRUZIONI D'USO / INSTRUCCIONES DE USO / NÁVOD NA POUŽITÍ / ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ / GEBRUIKSAANWIJZING / INSTRUÇÕES DE UTILIZAÇÃO / INSTRUKCJA UŻYCIA

ENGLISH

Description:

ConvaMax™ Superabsorber adhesive dressing with a silicone layer is a sterile dressing for use on highly exuding wounds, consisting of a breathable and waterproof nonwoven layer, a Superabsorber pad layer, a fabric wicking layer and soft silicone wound contact layer. The multi-layered construction facilitates fluid management to provide the optimal moist wound environment which leads to the promotion of faster wound closure. The absorbent pad provides superior absorption and locks exudates away. This minimizes the risk of maceration and damage to peri-wound skin.

Indications:

ConvaMax™ Superabsorber adhesive dressing is indicated for management of moderate to highly exuding wounds, including: leg ulcers, pressure ulcers, diabetic foot ulcers and dehisced surgical wounds.

Directions for use:

- After the wound is cleansed, the appropriate size dressing is selected to allow for sufficient space to the wound edge.
- ConvaMax™ Superabsorber adhesive dressing can be used both as a primary dressing or as a secondary dressing. When used as a primary dressing, ConvaMax™ Superabsorber adhesive dressing must be in close contact with the wound bed to ensure wound fluid is absorbed. White contact layer should be in contact with the skin and blue waterproof backing away from the body.
- ConvaMax™ Superabsorber adhesive dressing is self-adhesive but may require additional support with a fixation dressing or mild-to-moderate compression bandaging.
- The frequency of changing dressing depends on the level of exudate.
- Dressing removal can be facilitated by moistening the dressing with saline solution.
- Dispose of as clinical waste. If contaminated, follow local regulations

Contraindications:

- Do not use this product on dry or low exuding wounds, eyes, mucous membranes or in wound cavities because the dressing swells during absorption. Do not use on third-degree burns.
- ConvaMax™ Superabsorber adhesive dressing should not be used on patients with a known allergy or hypersensitivity to the dressing or its components.
- Arterial bleeds and heavily bleeding wounds.

Caution:

- Do not reuse. Do not use the dressing if the sterilized packaging is damaged or open.
- Do not cut the dressing, the cover must remain intact.

Storage: Please store the product in cool and dry place and avoid light

Shelf life: 3 years from date of manufacture

© 2021 ConvaTec Inc.

®/™ indicates a trademark of ConvaTec Inc.

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ConvaTec Inc. has granted a license to Huizhou Foryou Medical Devices Co., Ltd. to use the ConvaMax and ConvaTec trademarks.
®/™ oznacza znak towarowy ConvaTec Inc.
Shanghai International Holding Corp. GmbH (Europe) Eiffestrasse 80 20537 Hamburg, Germany
Distributed by: ConvaTec Limited First Avenue, Deeside Industrial Park Deeside, Flintshire, CH5 2NU, UK
ConvaTec Inc. Greensboro, NC 27409 USA
Importer: ConvaTec International Services GmbH Mühhlentalstrasse 36/38 CH-8200 Schaffhausen